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**TEMPERATURA: 37 °C**

TEST NAME	AU
DECIMAL	1
MEASURE UNIT	mg/dL
REACTION TYPE	END POINT
SAMPLE VOLUME (µL)	8
REAGENT VOLUME (µL)	
REAGENT 1	400
REAGENT 2	0
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	300
LINEARITY LIMIT	20.0
CONTAMINANTING	NO
REAGENT BLANKING	YES
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	510
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.00
MIX 2	0.00
INCUBATION 1	300
INCUBATION 2	0
LAG PHASE	3
MEASURE	1
MEASUREMENT TYPE	
CALIBRATE	STANDARD #
FACTOR	0.0

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TEST NAME	AEO
DECIMAL	0
MEASURE UNIT	UI/mL
REACTION TYPE	END POINT
SAMPLE VOLUME (µL)	4
REAGENT VOLUME (µL)	
REAGENT 1	400
REAGENT 2	0
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	2.000
LINEARITY LIMIT	800
CONTAMINANTING	NO
REAGENT BLANKING	YES
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	546
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.2
MIX 2	0.00
INCUBATION 1	180
INCUBATION 2	0
LAG PHASE	2
MEASURE	1
MEASUREMENT TYPE	
CALIBRATE	STANDARD *
FACTOR	

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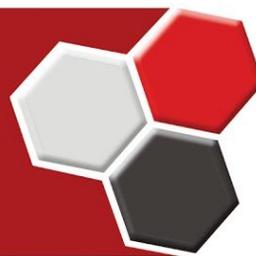
**TEMPERATURA: 37 °C**

TEST NAME	ALB
DECIMAL	1
MEASURE UNIT	g/dL
REACTION TYPE	END POINT
SAMPLE VOLUME (µL)	4
REAGENT VOLUME (µL)	
REAGENT 1	400
REAGENT 2	0
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	400
LINEARITY LIMIT	6.0
CONTAMINATING	YES
REAGENT BLANKING	YES
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	620
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.00
MIX 2	0.00
INCUBATION 1	120
INCUBATION 2	0
LAG PHASE	3
MEASURE	1
MEASUREMENT TYPE	
CALIBRATE	STANDARD #
FACTOR	0

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**TEMPERATURA: 37 °C**

TEST NAME	ALT
DECIMAL	0
MEASURE UNIT	U/L
REACTION TYPE	KINETIC
SAMPLE VOLUME (µL)	40
REAGENT VOLUME (µL)	
REAGENT 1	400
REAGENT 2	0
ABSORBANCE RANGE (mABS)	
MINIMUM	1000
MAXIMUM	2000
LINEARITY LIMIT	400
CONTAMINATING	NO
REAGENT BLANKING	NO
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	340
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.00
MIX 2	0.00
INCUBATION 1	60
INCUBATION 2	0
LAG PHASE	15
MEASURE	30
MEASUREMENT TYPE	
CALIBRATE	STANDARD #1
FACTOR	#2 = -1746

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**TEMPERATURA: 37 °C**

TEST NAME	AMILASE
DECIMAL	0
MEASURE UNIT	U/L
REACTION TYPE	KINETIC
SAMPLE VOLUME (µL)	8
REAGENT VOLUME (µL)	
REAGENT 1	400
REAGENT 2	0
ABSORBANCE RANGE (mABS)	
MINIMUM	000
MAXIMUM	500
LINEARITY LIMIT	2000
CONTAMINANTING	NO
REAGENT BLANKING	NO
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	405
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.00
MIX 2	0.00
INCUBATION 1	60
INCUBATION 2	0
LAG PHASE	15
MEASURE	30
MEASUREMENT TYPE	
CALIBRATE	STANDARD #1
FACTOR	#2 = 3953

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**TEMPERATURA: 37 °C**

TEST NAME	AST
DECIMAL	1
MEASURE UNIT	U/L
REACTION TYPE	KINETIC
SAMPLE VOLUME (µL)	40
REAGENT VOLUME (µL)	
REAGENT 1	400
REAGENT 2	0
ABSORBANCE RANGE (mABS)	
MINIMUM	1000
MAXIMUM	2000
LINEARITY LIMIT	400
CONTAMINATING	NO
REAGENT BLANKING	NO
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	340
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.00
MIX 2	0.00
INCUBATION 1	60
INCUBATION 2	0
LAG PHASE	15
MEASURE	30
MEASUREMENT TYPE	
CALIBRATE	STANDARD #1
FACTOR	#2 = -1746

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**TEMPERATURA: 37 °C**

TEST NAME	BD
DECIMAL	1
MEASURE UNIT	mg/dL
REACTION TYPE	END POINT
SAMPLE VOLUME (µL)	30
REAGENT VOLUME (µL)	
REAGENT 1	300
REAGENT 2	75
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	300
LINEARITY LIMIT	12
CONTAMINATING	NO
REAGENT BLANKING	NO
DIFFERENTIAL	YES (variável)
FILTERS (nm)	
FILTER 1	546
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.0
MIX 2	0.40
INCUBATION 1	180
INCUBATION 2	300
LAG PHASE	3
MEASURE	1
MEASUREMENT TYPE	
CALIBRATE	STANDARD #
FACTOR	0

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**TEMPERATURA: 37 °C**

TEST NAME	BT
DECIMAL	1
MEASURE UNIT	mg/dL
REACTION TYPE	END POINT
SAMPLE VOLUME (µL)	19
REAGENT VOLUME (µL)	
REAGENT 1	300
REAGENT 2	75
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	300
LINEARITY LIMIT	30
CONTAMINATING	NO
REAGENT BLANKING	NO
DIFFERENTIAL	YES (variável)
FILTERS (nm)	
FILTER 1	546
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.0
MIX 2	0.40
INCUBATION 1	180
INCUBATION 2	300
LAG PHASE	3
MEASURE	1
MEASUREMENT TYPE	
CALIBRATE	STANDARD #
FACTOR	0

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**TEMPERATURA: 37 °C**

TEST NAME	BD
DECIMAL	1
MEASURE UNIT	mg/dL
REACTION TYPE	END POINT
SAMPLE VOLUME (µL)	24
REAGENT VOLUME (µL)	
REAGENT 1	360
REAGENT 2	40
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	300
LINEARITY LIMIT	15
CONTAMINATING	NO
REAGENT BLANKING	NO
DIFFERENTIAL	YES
FILTERS (nm)	
FILTER 1	546
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.0
MIX 2	0.40
INCUBATION 1	30
INCUBATION 2	300
LAG PHASE	3
MEASURE	1
MEASUREMENT TYPE	
CALIBRATE	STANDARD #
FACTOR	0

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**TEMPERATURA: 37 °C**

TEST NAME	BT
DECIMAL	1
MEASURE UNIT	mg/dL
REACTION TYPE	END POINT
SAMPLE VOLUME (µL)	24
REAGENT VOLUME (µL)	
REAGENT 1	360
REAGENT 2	40
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	150
LINEARITY LIMIT	15
CONTAMINATING	NO
REAGENT BLANKING	NO
DIFFERENTIAL	YES
FILTERS (nm)	
FILTER 1	546
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.0
MIX 2	0.40
INCUBATION 1	30
INCUBATION 2	300
LAG PHASE	3
MEASURE	1
MEASUREMENT TYPE	
CALIBRATE	STANDARD #
FACTOR	0

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**TEMPERATURA: 37 °C**

TEST NAME	CA
DECIMAL	1
MEASURE UNIT	mg/dL
REACTION TYPE	END POINT
SAMPLE VOLUME (µL)	8
REAGENT VOLUME (µL)	
REAGENT 1	400
REAGENT 2	0
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	750
LINEARITY LIMIT	15.0
CONTAMINATING	YES
REAGENT BLANKING	YES
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	578
FILTER 2	620
TIMES (SEC)	
MIX 1	0.00
MIX 2	0.00
INCUBATION 1	180
INCUBATION 2	0
LAG PHASE	3
MEASURE	2
MEASUREMENT TYPE	
CALIBRATE	STANDARD #
FACTOR	0

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**TEMPERATURA: 37 °C**

TEST NAME	CA
DECIMAL	1
MEASURE UNIT	mg/dL
REACTION TYPE	END POINT
SAMPLE VOLUME (µL)	4
REAGENT VOLUME (µL)	
REAGENT 1	400
REAGENT 2	0
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	400
LINEARITY LIMIT	17.0
CONTAMINATING	YES
REAGENT BLANKING	YES
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	620
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.00
MIX 2	0.00
INCUBATION 1	60
INCUBATION 2	0
LAG PHASE	3
MEASURE	1
MEASUREMENT TYPE	
CALIBRATE	STANDARD #
FACTOR	0

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**TEMPERATURA: 37 °C**

TEST NAME	COLT
DECIMAL	0
MEASURE UNIT	mg/dL
REACTION TYPE	END POINT
SAMPLE VOLUME (µL)	4
REAGENT VOLUME (µL)	
REAGENT 1	400
REAGENT 2	0
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	300
LINEARITY LIMIT	500
CONTAMINANTING	NO
REAGENT BLANKING	YES
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	510
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.00
MIX 2	0.00
INCUBATION 1	600
INCUBATION 2	0
LAG PHASE	3
MEASURE	1
MEASUREMENT TYPE	
CALIBRATE	STANDARD #
FACTOR	0

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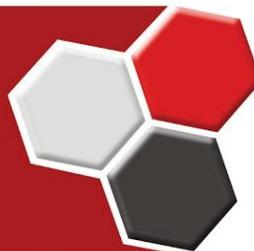
**TEMPERATURA: 37 °C**

TEST NAME	HDL
DECIMAL	0
MEASURE UNIT	mg/dL
REACTION TYPE	END POINT
SAMPLE VOLUME (µL)	40
REAGENT VOLUME (µL)	
REAGENT 1	400
REAGENT 2	0
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	300
LINEARITY LIMIT	100
CONTAMINATING	NO
REAGENT BLANKING	YES
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	510
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.00
MIX 2	0.00
INCUBATION 1	600
INCUBATION 2	0
LAG PHASE	3
MEASURE	1
MEASUREMENT TYPE	
CALIBRATE	STANDARD #
FACTOR	0

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**TEMPERATURA: 37 °C**

TEST NAME	CHE
DECIMAL	0
MEASURE UNIT	U/L
REACTION TYPE	KINETIC
SAMPLE VOLUME (µL)	4
REAGENT VOLUME (µL)	
REAGENT 1	200
REAGENT 2	50
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	3000
LINEARITY LIMIT	20000
CONTAMINATING	YES
REAGENT BLANKING	NO
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	405
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.20
MIX 2	0.20
INCUBATION 1	180
INCUBATION 2	15
LAG PHASE	120
MEASURE	180
MEASUREMENT TYPE	
CALIBRATE	STANDARD #1
FACTOR	#2 = - 68500

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**TEMPERATURA: 37 °C**

TEST NAME	CK
DECIMAL	0
MEASURE UNIT	U/L
REACTION TYPE	KINETIC
SAMPLE VOLUME (µL)	8
REAGENT VOLUME (µL)	
REAGENT 1	400
REAGENT 2	0
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	500
LINEARITY LIMIT	2000
CONTAMINATING	NO
REAGENT BLANKING	NO
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	340
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.00
MIX 2	0.00
INCUBATION 1	120
INCUBATION 2	0
LAG PHASE	15
MEASURE	30
MEASUREMENT TYPE	
CALIBRATE	STANDARD #1
FACTOR	#2 = 8095

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**TEMPERATURA: 37 °C**

TEST NAME	CK MB
DECIMAL	0
MEASURE UNIT	U/L
REACTION TYPE	KINETIC
SAMPLE VOLUME (µL)	20
REAGENT VOLUME (µL)	
REAGENT 1	400
REAGENT 2	0
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	500
LINEARITY LIMIT	500
CONTAMINATING	NO
REAGENT BLANKING	NO
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	340
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.00
MIX 2	0.00
INCUBATION 1	300
INCUBATION 2	0
LAG PHASE	15
MEASURE	120
MEASUREMENT TYPE	
CALIBRATE	STANDARD #1
FACTOR	#2 = 6667

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TEST NAME	CREAT
DECIMAL	2
MEASURE UNIT	mg/dL
REACTION TYPE	FIXED TIME
SAMPLE VOLUME (µL)	40
REAGENT VOLUME (µL)	
REAGENT 1	400
REAGENT 2	0
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	500
LINEARITY LIMIT	12.0
CONTAMINATING	NO
REAGENT BLANKING	NO
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	510
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.00
MIX 2	0.00
INCUBATION 1	30
INCUBATION 2	0
LAG PHASE	15
MEASURE	60
MEASUREMENT TYPE	
CALIBRATE	STANDARD #
FACTOR	0

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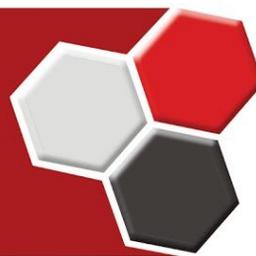
**TEMPERATURA: 37 °C**

TEST NAME	CREA
DECIMAL	1
MEASURE UNIT	mg/dL
REACTION TYPE	FIXED TIME
SAMPLE VOLUME (µL)	40
URINE (dil. 1/50)	40
REAGENT VOLUME (µL)	
REAGENT 1	400
REAGENT 2	0
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	500
LINEARITY LIMIT	10
CONTAMINATING	NO
REAGENT BLANKING	NO
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	510
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.00
MIX 2	0.00
INCUBATION 1	30
INCUBATION 2	0
LAG PHASE	15
MEASURE	60
MEASUREMENT TYPE	
CALIBRATE	STANDARD #
FACTOR	0

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**TEMPERATURA: 37 °C**

TEST NAME	FERRO
DECIMAL	0
MEASURE UNIT	µg/dL
REACTION TYPE	END POINT
SAMPLE VOLUME (µL)	25
REAGENT VOLUME (µL)	
REAGENT 1	200
REAGENT 2	50
ABSORBANCE RANGE (mABS)	
MINIMUM	0.0
MAXIMUM	2.000
LINEARITY LIMIT	1000
CONTAMINANTING	YES
REAGENT BLANKING	YES
DIFFERENTIAL	YES
FILTERS (nm)	
FILTER 1	546
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.00
MIX 2	0.20
INCUBATION 1	20
INCUBATION 2	300
LAG PHASE	1
MEASURE	1
MEASUREMENT TYPE	
CALIBRATE	STANDARD #
NORMAL RANGE	55-155

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**TEMPERATURA: 37 °C**

TEST NAME	FALC
DECIMAL	0
MEASURE UNIT	U/L
REACTION TYPE	KINETIC
SAMPLE VOLUME (µL)	8
REAGENT VOLUME (µL)	
REAGENT 1	400
REAGENT 2	0
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	1200
LINEARITY LIMIT	1500
CONTAMINANTING	YES
REAGENT BLANKING	NO
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	405
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.00
MIX 2	0.00
INCUBATION 1	60
INCUBATION 2	0
LAG PHASE	15
MEASURE	30
MEASUREMENT TYPE	
CALIBRATE	STANDARD #1
FACTOR	#2 = 2764

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**TEMPERATURA: 37 °C**

TEST NAME	FOSF
DECIMAL	1
MEASURE UNIT	mg/dL
REACTION TYPE	END POINT
SAMPLE VOLUME (µL)	4
URINE	-
REAGENT VOLUME (µL)	
REAGENT 1	400
REAGENT 2	0
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	1500
LINEARITY LIMIT	20
CONTAMINANTING	NO
REAGENT BLANKING	YES
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	340
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.00
MIX 2	0.00
INCUBATION 1	300
INCUBATION 2	0
LAG PHASE	3
MEASURE	1
MEASUREMENT TYPE	
CALIBRATE	STANDARD #
FACTOR	0

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TEST NAME	FRUT
DECIMAL	2
MEASURE UNIT	mmol/L
REACTION TYPE	FIXED TIME
SAMPLE VOLUME (µL)	20
REAGENT VOLUME (µL)	
REAGENT 1	400
REAGENT 2	0
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	500
LINEARITY LIMIT	7.0
CONTAMINATING	YES
REAGENT BLANKING	NO
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	535
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.00
MIX 2	0.00
INCUBATION 1	600
INCUBATION 2	0
LAG PHASE	10
MEASURE	300
MEASUREMENT TYPE	
CALIBRATE	STANDARD *
FACTOR	0

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**TEMPERATURA: 37 °C**

TEST NAME	GGT
DECIMAL	0
MEASURE UNIT	U/L
REACTION TYPE	KINETIC
SAMPLE VOLUME (µL)	20
REAGENT VOLUME (µL)	
REAGENT 1	400
REAGENT 2	0
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	750
LINEARITY LIMIT	700
CONTAMINATING	NO
REAGENT BLANKING	NO
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	405
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.00
MIX 2	0.00
INCUBATION 1	60
INCUBATION 2	0
LAG PHASE	15
MEASURE	30
MEASUREMENT TYPE	
CALIBRATE	STANDARD #1
FACTOR	#2 = 2764

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**TEMPERATURA: 37 °C**

TEST NAME	GLICOSE
DECIMAL	0
MEASURE UNIT	mg/dL
REACTION TYPE	END POINT
SAMPLE VOLUME (µL)	4
REAGENT VOLUME (µL)	
REAGENT 1	400
REAGENT 2	0
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	300
LINEARITY LIMIT	500
CONTAMINANTING	NO
REAGENT BLANKING	YES
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	510
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.00
MIX 2	0.00
INCUBATION 1	600
INCUBATION 2	0
LAG PHASE	3
MEASURE	2
MEASUREMENT TYPE	
CALIBRATE	STANDARD #
FACTOR	0

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**TEMPERATURA: 37 °C**

TEST NAME	HDL DIRETO
DECIMAL	0
MEASURE UNIT	mg/dL
REACTION TYPE	FIXED TIME
SAMPLE VOLUME (µL)	3
REAGENT VOLUME (µL)	
REAGENT 1	300
REAGENT 2	100
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	2000
LINEARITY LIMIT	150
CONTAMINATING	YES
REAGENT BLANKING	YES
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	578
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.20
MIX 2	0.20
INCUBATION 1	300
INCUBATION 2	5
LAG PHASE	3
MEASURE	120
MEASUREMENT TYPE	
CALIBRATE	STANDARD #
FACTOR	

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**TEMPERATURA: 37 °C**

TEST NAME	HDL DIRETO
DECIMAL	0
MEASURE UNIT	mg/dL
REACTION TYPE	FIXED TIME
SAMPLE VOLUME (µL)	3
REAGENT VOLUME (µL)	
REAGENT 1	300
REAGENT 2	100
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	2000
LINEARITY LIMIT	200
CONTAMINATING	YES
REAGENT BLANKING	YES
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	546
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.20
MIX 2	0.20
INCUBATION 1	300
INCUBATION 2	5
LAG PHASE	3
MEASURE	120
MEASUREMENT TYPE	
CALIBRATE	STANDARD #
FACTOR	

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TEST NAME	LDL DIRETO
DECIMAL	0
MEASURE UNIT	mg/dL
REACTION TYPE	END POINT
SAMPLE VOLUME (µL)	3
REAGENT VOLUME (µL)	
REAGENT 1	375
REAGENT 2	125
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	2000
LINEARITY LIMIT	450
CONTAMINATING	YES
REAGENT BLANKING	YES
DIFFERENTIAL	YES
FILTERS (nm)	
FILTER 1	546
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.00
MIX 2	0.20
INCUBATION 1	300
INCUBATION 2	300
LAG PHASE	2
MEASURE	2
MEASUREMENT TYPE	
CALIBRATE	STANDARD *
FACTOR	

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**TEMPERATURA: 37 °C**

TEST NAME	LDH
DECIMAL	0
MEASURE UNIT	U/L
REACTION TYPE	KINETIC
SAMPLE VOLUME (µL)	8
REAGENT VOLUME (µL)	
REAGENT 1	400
REAGENT 2	0
ABSORBANCE RANGE (mABS)	
MINIMUM	800
MAXIMUM	2000
LINEARITY LIMIT	1800
CONTAMINATING	NO
REAGENT BLANKING	NO
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	340
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.00
MIX 2	0.00
INCUBATION 1	60
INCUBATION 2	0
LAG PHASE	15
MEASURE	30
MEASUREMENT TYPE	
CALIBRATE	STANDARD #1
FACTOR	#2 = - 8095

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**TEMPERATURA: 37 °C**

TEST NAME	MG
DECIMAL	2
MEASURE UNIT	mg/dL
REACTION TYPE	END POINT
SAMPLE VOLUME (µL)	4
REAGENT VOLUME (µL)	
REAGENT 1	400
REAGENT 2	0
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	2400
LINEARITY LIMIT	4.5
CONTAMINANTING	NO
REAGENT BLANKING	YES
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	510
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.00
MIX 2	0.00
INCUBATION 1	120
INCUBATION 2	0
LAG PHASE	1
MEASURE	1
MEASUREMENT TYPE	
CALIBRATE	STANDARD #
FACTOR	0

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TEST NAME	ALB
DECIMAL	1
MEASURE UNIT	mg/L
REACTION TYPE	KINETIC
SAMPLE VOLUME (µL)	3
REAGENT VOLUME (µL)	
REAGENT 1	400
REAGENT 2	0
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	600
LINEARITY LIMIT	130
CONTAMINANTING	NO
REAGENT BLANKING	YES
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	546
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.00
MIX 2	0.00
INCUBATION 1	5
INCUBATION 2	0
LAG PHASE	90
MEASURE	30
MEASUREMENT TYPE	
CALIBRATE	STANDARD *
FACTOR	0.0

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**TEMPERATURA: 37 °C**

TEST NAME	PT
DECIMAL	1
MEASURE UNIT	g/dL
REACTION TYPE	END POINT
SAMPLE VOLUME (µL)	8
REAGENT VOLUME (µL)	
REAGENT 1	400
REAGENT 2	0
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	250
LINEARITY LIMIT	14.0
CONTAMINANTING	NO
REAGENT BLANKING	YES
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	546
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.00
MIX 2	0.00
INCUBATION 1	300
INCUBATION 2	0
LAG PHASE	3
MEASURE	1
MEASUREMENT TYPE	
CALIBRATE	STANDARD #
FACTOR	0

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**TEMPERATURA: 37 °C**

TEST NAME	PROTUR
DECIMAL	1
MEASURE UNIT	mg/dL
REACTION TYPE	END POINT
SAMPLE VOLUME (µL)	20
REAGENT VOLUME (µL)	
REAGENT 1	400
REAGENT 2	0
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	250
LINEARITY LIMIT	100
CONTAMINANTING	NO
REAGENT BLANKING	YES
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	620
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.00
MIX 2	0.00
INCUBATION 1	300
INCUBATION 2	0
LAG PHASE	1
MEASURE	1
MEASUREMENT TYPE	
CALIBRATE	STANDARD #
FACTOR	0

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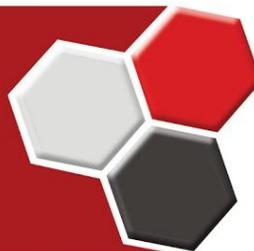
**TEMPERATURA: 37 °C**

TEST NAME	TRIG
DECIMAL	0
MEASURE UNIT	mg/dL
REACTION TYPE	END POINT
SAMPLE VOLUME (µL)	4
REAGENT VOLUME (µL)	
REAGENT 1	400
REAGENT 2	0
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	300
LINEARITY LIMIT	1000
CONTAMINATING	NO
REAGENT BLANKING	YES
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	510
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.00
MIX 2	0.00
INCUBATION 1	600
INCUBATION 2	0
LAG PHASE	3
MEASURE	1
MEASUREMENT TYPE	
CALIBRATE	STANDARD #
FACTOR	0

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**TEMPERATURA: 37 °C**

TEST NAME	UREUV
DECIMAL	0
MEASURE UNIT	mg/dL
REACTION TYPE	FIXED TIME
SAMPLE VOLUME (µL)	4
REAGENT VOLUME (µL)	
REAGENT 1	400
REAGENT 2	0
ABSORBANCE RANGE (mABS)	
MINIMUM	1000
MAXIMUM	2000
LINEARITY LIMIT	300
CONTAMINATING	NO
REAGENT BLANKING	NO
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	340
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.00
MIX 2	0.00
INCUBATION 1	30
INCUBATION 2	0
LAG PHASE	10
MEASURE	60
MEASUREMENT TYPE	
CALIBRATE	STANDARD #
FACTOR	0

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**TEMPERATURA: 37 °C**

TEST NAME	UREIA
DECIMAL	0
MEASURE UNIT	mg/dL
REACTION TYPE	END POINT
SAMPLE VOLUME (µL)	3
REAGENT VOLUME (µL)	
REAGENT 1	300
REAGENT 2	300
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	2000
LINEARITY LIMIT	300
CONTAMINATING	YES
REAGENT BLANKING	YES
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	620
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.00
MIX 2	0.20
INCUBATION 1	300
INCUBATION 2	300
LAG PHASE	3
MEASURE	1
MEASUREMENT TYPE	
CALIBRATE	STANDARD #
FACTOR	0

EDIÇÃO: 09/11

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© AUTOHUMALYSER 900S é marca registrada de seus proprietários.