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Test	AU		
Type	End Pt.	Primary Wavelength	500
Unit	mg/dL	High Wavelength	---
Number of decimals	1	Low Wavelength	---
Diluent	99 Diluent	Systematic serum blank	NO
Pre-washing	NO	T Tolerance Factor	0.0200
First reagent vol. (µl)	200	Antigen Excess test (%)	NO
Second reagent vol. (µl)	0	Slope	1.00
Sample Volume (µl)	4.0	Intercept	0.00
Lower normal value	@		
Higher normal value	@		
Systematic dilution	1		
Dilution For Urine	10		
Low Reagent Blank	0.0000	Reaction delay	05:00
High Reagent Blank	0.2000	Incubation	00:00
Maximum Deviation	0.0200	Total	05:00

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Test	AEO		
Type	Kin. 1	Primary Wavelength	550
Unit	UI/mL	High Wavelength	---
Number of decimals	0	Low Wavelength	---
Diluent	99 Diluent	Systematic serum blank	NO
Pre-washing	NO		
First reagent vol. (µl)	300	Depletion Limit	1.0000
Second reagent vol. (µl)	0	T Tolerance Factor	0.5000
Sample Volume (µl)	3.0	C Compensation Factor	1.0000
Lower normal value	0	Slope	1.00
Higher normal value	200	Intercept	0.00
Dilution factor	3		
Systematic dilution	1	Reaction delay	00:22
Dilution For Urine	1	Incubation	02:34
		Total	02:56
Low Reagent Blank	0.0000		
High Reagent Blank	2.0000		
Maximum Deviation	0.2000		

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Test	ALB		
Type	End Pt.	Primary Wavelength	600
Unit	g/dL	High Wavelength	---
Number of decimals	2	Low Wavelength	---
Diluent	99 Diluent	Systematic serum blank	NO
Pre-washing	NO	T Tolerance Factor	0.0150
First reagent vol. (µl)	300	Antigen Excess test (%)	NO
Second reagent vol. (µl)	0		
Sample Volume (µl)	3.0		
Lower normal value	@	Slope	1.00
Higher normal value	@	Intercept	0.00
Systematic dilution	1	Reaction delay	02:00
Dilution For Urine	1		
Low Reagent Blank	0.0000		
High Reagent Blank	0.1000		
Maximum Deviation	0.0150		

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Test	ALT	Primary Wavelength	340
Type	Kin. 0		
Unit	U/L	High Wavelength	---
Number of decimals	0	Low Wavelength	---
Diluent	99 Diluent	Systematic serum blank	NO
Pre-washing	NO	High limit	0.0100
First reagent vol. (μ l)	200	Depletion limit	0.1040
Second reagent vol. (μ l)	0		
Sample Volume (μ l)	20.0	C Compensation Factor	0.0001
Lower normal value	@	Slope	1.00
Higher normal value	@	Intercept	0.00
Dilution factor	10		
Systematic dilution	1	Reaction delay	01:00
Dilution For Urine	1		
Low Reagent Blank	0.4000		
High Reagent Blank	3.5000		
Maximum Deviation	0.0500		

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Test Type	AMI Kin. 0	Primary Wavelength	405
Unit	U/L	High Wavelength	---
Number of decimals	0	Low Wavelength	---
Diluent	99 Diluent	Systematic serum blank	NO
Pre-washing	NO	High limit	0.0100
First reagent vol. (μ l)	200	Depletion limit	0.2400
Second reagent vol. (μ l)	0		
Sample Volume (μ l)	4.0	C Compensation Factor	0.0001
Lower normal value	@	Slope	1.00
Higher normal value	@	Intercept	0.00
Dilution factor	10		
Systematic dilution	1	Reaction delay	01:00
Dilution For Urine	1		
Low Reagent Blank	0.0000		
High Reagent Blank	0.4000		
Maximum Deviation	0.0100		

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Test	AST		
Type	Kin. 0	Primary Wavelength	340
Unit	U/L	High Wavelength	
Number of decimals	0	Low Wavelength	---
Diluent	99 Diluent	Systematic serum blank	NO
Pre-washing	NO	High limit	0.0100
First reagent vol. (μ l)	200	Depletion limit	0.1040
Second reagent vol. (μ l)	0		
Sample Volume (μ l)	20.0	C Compensation Factor	1.0000
Lower normal value	@	Slope	1.00
Higher normal value	@	Intercept	0.00
Dilution factor	10		
Systematic dilution	1	Reaction delay	01:00
Dilution For Urine	1		
Low Reagent Blank	0.4000		
High Reagent Blank	3.5000		
Maximum Deviation	0.0500		

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Test	BD		
Type	End Pt.	Primary Wavelength	550
Unit	mg/dL	High Wavelength	---
Number of decimals	2	Low Wavelength	---
Diluent	99 Diluent	Systematic serum blank	NO
Pre-washing	NO	T Tolerance Factor	0.0100
First reagent vol. (µl)	180	Antigen Excess test (%)	NO
Second reagent vol. (µl)	45		
Sample Volume (µl)	18.0		
Lower normal value	@	Slope	1.00
Higher normal value	@	Intercept	0.00
Systematic dilution	1	Delay for Reagt 2 Dispense	01:48
Dilution For Urine	1	Reaction delay	03:00
Low Reagent Blank	0.0000	Total	03:48
High Reagent Blank	0.1000		
Maximum Deviation	0.0300		

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Test	BT		
Type	End Pt.	Primary Wavelength	550
Unit	mg/dL	High Wavelength	---
Number of decimals	2	Low Wavelength	---
Diluent	99 Diluent	Systematic serum blank	NO
Pre-washing	NO	T Tolerance Factor	0.0100
First reagent vol. (µl)	180	Antigen Excess test (%)	NO
Second reagent vol. (µl)	45		
Sample Volume (µl)	11.0		
Lower normal value	@	Slope	1.00
Higher normal value	@	Intercept	0.00
Systematic dilution	1	Delay for Reagt 2 Dispense	01:48
Dilution For Urine	1	Reaction delay	03:00
Low Reagent Blank	0.0000	Total	03:48
High Reagent Blank	0.1000		
Maximum Deviation	0.0300		

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Test	CA		
Type	End Pt.	Primary Wavelength	600
Unit	mg/dL	High Wavelength	---
Number of decimals	2	Low Wavelength	---
Diluent	99 Diluent	Systematic serum blank	NO
Pre-washing	NO	T Tolerance Factor	0.0150
First reagent vol. (µl)	300	Antigen Excess test (%)	NO
Second reagent vol. (µl)	0	Slope	1.00
Sample Volume (µl)	3.0	Intercept	0.00
Lower normal value	@	Reaction Delay	01:00
Higher normal value	@		
Systematic dilution	1		
Dilution For Urine	1		
Low Reagent Blank	0.0000	Reaction delay	01:00
High Reagent Blank	3.5000	Incubation	00:00
Maximum Deviation	0.0150	Total	01:36

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Test	CALCIO		
Type	End Pt.	Primary Wavelength	550
Unit	mg/dL	High Wavelength	---
Number of decimals	1	Low Wavelength	---
Diluent	99 Diluent	Systematic serum blank	NO
Pre-washing	NO	T Tolerance Factor	0.0100
First reagent vol. (µl)	150	Antigen Excess test (%)	NO
Second reagent vol. (µl)	50		
Sample Volume (µl)	4.0		
Lower normal value	@	Slope	1.00
Higher normal value	@	Intercept	0.00
Systematic dilution	1	Delay for Reagt 2 Dispense	00:48
Dilution For Urine	1	Reaction Delay	02:00
Low Reagent Blank	0.0000		
High Reagent Blank	0.7500		
Maximum Deviation	0.0100		

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Test	CLF		
Type	End Pt.	Primary Wavelength	550
Unit	µg/dL	High Wavelength	---
Number of decimals	0	Low Wavelength	---
Diluent	99 Diluent	Systematic serum blank	NO
Pre-washing	NO	T Tolerance Factor	0.0100
First reagent vol. (µl)	220	Antigen Excess test (%)	NO
Second reagent vol. (µl)	60		
Sample Volume (µl)	30		
Lower normal value	@	Slope	1.00
Higher normal value	@	Intercept	0.00
Systematic dilution	1	Delay for Reagt 2 Dispense	00:48
Dilution For Urine	1	Reaction Delay	04:00
Low Reagent Blank	0.0000		
High Reagent Blank	2.0000		
Maximum Deviation	0.0100		

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Test	Cl		
Type	End Pt.	Primary Wavelength	500
Unit	mEq/L	High Wavelength	---
Number of decimals	0	Low Wavelength	---
Diluent	99 Diluent	Systematic serum blank	NO
Pre-washing	NO	T Tolerance Factor	0.0100
First reagent vol. (µl)	387	Antigen Excess test (%)	NO
Second reagent vol. (µl)	0		
Sample Volume (µl)	2.0		
Lower normal value	@	Slope	1.00
Higher normal value	@	Intercept	0.00
Systematic dilution	1	Reaction delay	01:00
Dilution For Urine	1	Incubation	00:00
		Total	
Low Reagent Blank	0.0000		
High Reagent Blank	0.1500		
Maximum Deviation	0.0200		

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Test	COLEST		
Type	End Pt.	Primary Wavelength	500
Unit	mg/dL	High Wavelength	---
Number of decimals	0	Low Wavelength	---
Diluent	99 Diluent	Systematic serum blank	NO
Pre-washing	NO	T Tolerance Factor	0.0200
First reagent vol. (µl)	300	Antigen Excess test (%)	NO
Second reagent vol. (µl)	0		
Sample Volume (µl)	3.0		
Lower normal value	@	Slope	1.00
Higher normal value	@	Intercept	0.00
Systematic dilution	1	Reaction delay	05:00
Dilution For Urine	1	Incubation	00:00
		Total	05:00
Low Reagent Blank	0.0000		
High Reagent Blank	0.1500		
Maximum Deviation	0.0200		

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Test	COHDL		
Type	End Pt.	Primary Wavelength	500
Unit	mg/dL	High Wavelength	---
Number of decimals	0	Low Wavelength	---
Diluent	99 Diluent	Systematic serum blank	NO
Pre-washing	NO	T Tolerance Factor	0.0200
First reagent vol. (µl)	200	Antigen Excess test (%)	NO
Second reagent vol. (µl)	0		
Sample Volume (µl)	20.0		
Lower normal value	*	Slope	1.00
Higher normal value	*	Intercept	0.00
Systematic dilution	1	Reaction delay	05:00
Dilution For Urine	1	Incubation	00:00
		Total	05:00
Low Reagent Blank	0.0000		
High Reagent Blank	0.1500		
Maximum Deviation	0.0200		

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COLINESTERASE – PP - Cat. 415M
COLINESTERASE – PP - Cat. 415

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Test	CHE		
Type	Kin. 0	Primary Wavelength	405
Unit	U/L	High Wavelength	---
Number of decimals	0	Low Wavelength	---
Diluent	99 Diluent	Systematic serum blank	NO
Pre-washing	NO	High limit	0.0100
First reagent vol. (µl)	200	Depletion limit	0.1170
Second reagent vol. (µl)	50		
Sample Volume (µl)	4.0	DT Tolerance Factor	0.0300
Lower normal value	@	Slope	1.00
Higher normal value	@	Intercept	0.00
Dilution factor	2		
Systematic dilution	1	Delay for Reagt. 2 dispense	03:00
		Reaction delay	03:00
Dilution For Urine	10	Incubation	02:00
		Total	08:00
Low Reagent Blank	0.4000		
High Reagent Blank	3.5000		
Maximum Deviation	0.0500		

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Test	CKMB		
Type	Kin. 0	Primary Wavelength	340
Unit	U/L	High Wavelength	---
Number of decimals	0	Low Wavelength	---
Diluent	99 Diluent	Systematic serum blank	NO
Pre-washing	NO	High limit	0.0100
First reagent vol. (μ l)	200	Depletion limit	0.1170
Second reagent vol. (μ l)	0		
Sample Volume (μ l)	10	C Compensation Factor	0.0001
Lower normal value	@	Slope	1.00
Higher normal value	@	Intercept	0.00
Dilution factor	10		
Systematic dilution	1	Reaction delay	04:36
Dilution For Urine	1		
Low Reagent Blank	0.0000		
High Reagent Blank	0.3000		
Maximum Deviation	0.0100		

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Test	CK-NAC		
Type	Kin. 0	Primary Wavelength	340
Unit	U/L	High Wavelength	---
Number of decimals	0	Low Wavelength	---
Diluent	99 Diluent	Systematic serum blank	NO
Pre-washing	NO	High limit	0.0100
First reagent vol. (μ l)	200	Depletion limit	0.1170
Second reagent vol. (μ l)	0		
Sample Volume (μ l)	4.0	C Compensation Factor	0.0001
Lower normal value	@	Slope	1.00
Higher normal value	@	Intercept	0.00
Dilution factor	10		
Systematic dilution	1	Reaction delay	02:00
Dilution For Urine	1		
Low Reagent Blank	0.0000		
High Reagent Blank	0.3000		
Maximum Deviation	0.0100		

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Test	CREAT		
Type	Kin. 1	Primary Wavelength	500
Unit	mg/dL	High Wavelength	---
Number of decimals	2	Low Wavelength	---
Diluent	99 Diluent	Systematic serum blank	NO
Pre-washing	NO		
First reagent vol. (µl)	200	Depletion Limit	3.5000
Second reagent vol. (µl)	0	T Tolerance Factor	1.0000
Sample Volume (µl)	10.0	C Compensation Factor	1.0000
Lower normal value	@	Slope	1.00
Higher normal value	@	Intercept	0.00
Dilution factor	4		
Systematic dilution	1	Reaction delay	00:36
Dilution For Urine	10	Incubation	03:00
		Total	03:36
Low Reagent Blank	0.1000		
High Reagent Blank	0.2500		
Maximum Deviation	0.0200		

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Test	Ferro		
Type	End Pt.	Primary Wavelength	550
Unit	µg/dL	High Wavelength	---
Number of decimals	0	Low Wavelength	---
Diluent	99 Diluent		
Pre-washing	NO		
First reagent vol. (µl)	200	Depletion Limit	
Second reagent vol. (µl)	50	T Tolerance Factor	0.0100
Sample Volume (µl)	25	C Compensation Factor	1.0000
Lower normal value	@	Slope	1.00
Higher normal value	@	Intercept	0.00
Dilution factor		2 nd Reagent dispense delay	00:48
Systematic dilution	1	Reaction delay	04:00
Dilution For Urine	1		
Low Reagent Blank	0.0000		
High Reagent Blank	1.0000		
Maximum Deviation	0.0100		

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Test	FALC		
Type	Kin. 0	Primary Wavelength	405
Unit	U/L	High Wavelength	---
Number of decimals	0	Low Wavelength	---
Diluent	99 Diluent	Systematic serum blank	NO
Pre-washing	NO	High limit	0.0100
First reagent vol. (μ l)	200	Depletion limit	0.2560
Second reagent vol. (μ l)	0		
Sample Volume (μ l)	4.0	C Compensation Factor	0.0001
Lower normal value	@	Slope	1.00
Higher normal value	@	Intercept	0.00
Dilution factor	10		
Systematic dilution	1	Reaction delay	01:00
Dilution For Urine	1		
Low Reagent Blank	0.0000		
High Reagent Blank	0.6000		
Maximum Deviation	0.0100		

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Test	POSF		
Type	ENDP	Primary Wavelength	340
Unit	mg/dL	High Wavelength	-
Number of decimals	1	Low Wavelength	-
Diluent	99 Diluent	Polychromatic Factor:	
		Systematic serum blank	NO
		Serum Factor:	
		High Limit:	
		Depletion Limit	3.5000
Pre-washing	NO	T Tolerance Factor	1.0000
First reagent vol. (µl)	300	Antigen Excess test (%)	NO
Second reagent vol. (µl)	0	C. Compensation Factor	1.0000
Sample Volume (µl)	3.0		
Lower normal value	@	Slope	1.00
Higher normal value	@	Intercept	0.00
Dilution Factor		Delay for reagt.2 disp:	
Systematic dilution	1	Reaction delay	00:12
Dilution For Urine	10	Incubation	05:00
		Total:	05:12
Low Reagent Blank	0.0000		
High Reagent Blank	0.7500		
Maximum Deviation	0.0100		

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Test	FRUT		
Type	Kin. 1	Primary Wavelength	550
Unit	mmol/L	High Wavelength	---
Number of decimals	2	Low Wavelength	---
Diluent	99 Diluent	Systematic serum blank	NO
Pre-washing	NO		
First reagent vol. (µl)	200	Depletion Limit	0.3000
Second reagent vol. (µl)	0	T Tolerance Factor	1.0000
Sample Volume (µl)	10.0	C Compensation Factor	1.0000
Lower normal value	2.00	Slope	1.00
Higher normal value	2.80	Intercept	0.00
Dilution factor	2		
Systematic dilution	1	Reaction delay	05:00
Dilution For Urine	1	Incubation	03:36
		Total	08:36
Low Reagent Blank	0.0000		
High Reagent Blank	2.0000		
Maximum Deviation	0.0200		

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Usar o valor específico de seu aparelho (Ver etiqueta).

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Test	GGT		
Type	Kin. 0	Primary Wavelength	405
Unit	U/L	High Wavelength	---
Number of decimals	0	Low Wavelength	---
Diluent	99 Diluent	Systematic serum blank	NO
Pre-washing	NO	High limit	0.0100
First reagent vol. (µl)	200	Depletion limit	0.1560
Second reagent vol. (µl)	0		
Sample Volume (µl)	10.0	C Compensation Factor	0.0001
Lower normal value	@	Slope	1.00
Higher normal value	@	Intercept	0.00
Dilution factor	10		
Systematic dilution	1	Reaction delay	01:00
Dilution For Urine	1		
Low Reagent Blank	0.0000		
High Reagent Blank	0.6000		
Maximum Deviation	0.0100		

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Test	Gluc		
Type	Kin 1	Primary Wavelength	500
Unit	mg/dL	High Wavelength	---
Number of decimals	0	Low Wavelength	---
Diluent	99 Diluent	Systematic serum blank	NO
		Depletion Limit	3.5000
Pre-washing	NO	T Tolerance Factor	1.0000
First reagent vol. (µl)	300	C Compensation Factor	1.0000
Second reagent vol. (µl)	0		
Sample Volume (µl)	3.0		
Lower normal value	@	Slope	1.00
Higher normal value	@	Intercept	0.00
Systematic dilution	1	Reaction delay	00:24
Dilution For Urine	1	Incubation	04:00
		Total	04:24
Low Reagent Blank	0.0000		
High Reagent Blank	0.1500		
Maximum Deviation	0.0100		

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Test	HDL DIR		
Type	End Pt.	Primary Wavelength	600
Unit	mg/dL	High Wavelength	---
Number of decimals	0	Low Wavelength	---
Diluent	99 Diluent	Systematic Serum Blank	NO
Pre-washing	NO		
		DT Tolerance Factor	0.0300
First reagent vol. (µl)	250	Slope	1.00
		Intercept	0.0
Second reagent vol. (µl)	80		
Sample Volume (µl)	3.0		
Lower normal value	@		
Higher normal value	@	Delay for Feagt 2 Diapense	05:00
Dilution Factor	2	Reaction Delay	05:00
Systematic dilution	1	Incubation	00:00
Dilution for Urine	10	Total	05:00
Low Reagent Blank	0.0000		
High Reagent Blank	0.3000		
Maximum Deviation	0.0300		

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Test	LDL DIR		
Type	End Pt.	Primary Wavelength	550
Unit	mg/dL	High Wavelength	---
Number of decimals	0	Low Wavelength	---
Diluent	99 Diluent	Depletion Limit	1.0000
Pre-washing	NO	T Tolerance Factor	0.0200
First reagent vol. (µl)	180	C Compesation Factor	1.000
Second reagent vol. (µl)	60	Slope	1.00
Sample Volume (µl)	3.0	Intercept	0.00
Lower normal value			
Higher normal value	130		
Dilution Factor	1.0		
Systematic dilution	1		
Low Reagent Blank	0.0000	2 nd Reagente dispense delay	01:48
High Reagent Blank	1.5000	Reaction time	02:36
Maximum Deviation	0.0500		

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Usar o valor específico de seu aparelho (ver etiqueta).

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Test	LDH UV		
Type	Kin. 0	Primary Wavelength	340
Unit	U/L	High Wavelength	---
Number of decimals	0	Low Wavelength	---
Diluent	99 Diluent	Systematic serum blank	NO
Pre-washing	NO	High limit	0.0100
First reagent vol. (μ l)	200	Depletion limit	0.1050
Second reagent vol. (μ l)	0		
Sample Volume (μ l)	4.0	C Compensation Factor	0.0001
Lower normal value	@	Slope	1.00
Higher normal value	@	Intercept	0.00
Dilution factor	10		
Systematic dilution	1	Reaction delay	01:00
Dilution For Urine	1		
Low Reagent Blank	0.4000		
High Reagent Blank	3.5000		
Maximum Deviation	0.0100		

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Test	MG		
Type	End Pt.	Primary Wavelength	500
Unit	mg/dL	High Wavelength	---
Number of decimals	2	Low Wavelength	---
Diluent	99 Diluent	Systematic serum blank	NO
Pre-washing	NO	T Tolerance Factor	0.0100
First reagent vol. (µl)	300	Antigen Excess test (%)	NO
Second reagent vol. (µl)	0		
Sample Volume (µl)	3.0		
Lower normal value	@	Slope	1.00
Higher normal value	@	Intercept	0.00
Systematic dilution	1	Reaction delay	03:00
Dilution For Urine	5		
Low Reagent Blank	0.0000		
High Reagent Blank	0.4000		
Maximum Deviation	0.0100		

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Test	PCR		
Type	Kin. 1	Primary Wavelength	550
Unit	mg/L	High Wavelength	---
Number of decimals	2	Low Wavelength	---
Diluent	99 Diluent	Systematic serum blank	NO
Pre-washing	NO		
First reagent vol. (µl)	300	Depletion Limit	2.0000
Second reagent vol. (µl)	0	T Tolerance Factor	0.5000
Sample Volume (µl)	2.0	C Compensation Factor	1.0000
Lower normal value	0.00	Slope	1.00
Higher normal value	5.00	Intercept	0.00
Dilution factor	1		
Systematic dilution	1	Reaction delay	00:22
Dilution For Urine	1	Incubation	02:34
		Total	02:56
Low Reagent Blank	0.0000		
High Reagent Blank	1.8000		
Maximum Deviation	0.2000		

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Test	PT		
Type	End Pt.	Primary Wavelength	550
Unit	g/dL	High Wavelength	---
Number of decimals	1	Low Wavelength	---
Diluent	99 Diluent	Systematic serum blank	NO
Pre-washing	NO	T Tolerance Factor	0.0100
First reagent vol. (µl)	250	Antigen Excess test (%)	NO
Second reagent vol. (µl)	0		
Sample Volume (µl)	5.0		
Lower normal value	@	Slope	1.00
Higher normal value	@	Intercept	0.00
Systematic dilution	1	Reaction delay	04:48
Dilution For Urine	1		
Low Reagent Blank	0.0000		
High Reagent Blank	0.2000		
Maximum Deviation	0.0100		

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Test	PU		
Type	End Pt.	Primary Wavelength	600
Unit	mg/dL	High Wavelength	---
Number of decimals	1	Low Wavelength	---
Diluent	99 Diluent	Systematic serum blank	NO
Pre-washing	NO	T Tolerance Factor	0.0100
First reagent vol. (µl)	200	Antigen Excess test (%)	NO
Second reagent vol. (µl)	0		
Sample Volume (µl)	10		
Lower normal value	@	Slope	1.00
Higher normal value	@	Intercept	0.00
Systematic dilution	1	Reaction delay	04:24
Dilution For Urine	1		
Low Reagent Blank	0.0800		
High Reagent Blank	0.1500		
Maximum Deviation	0.0100		

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Test	TRIG		
Type	End Pt.	Primary Wavelength	500
Unit	mg/dL	High Wavelength	---
Number of decimals	0	Low Wavelength	---
Diluent	99 Diluent	Systematic serum blank	NO
Pre-washing	NO	T Tolerance Factor	0.0100
First reagent vol. (µl)	300	Antigen Excess test (%)	NO
Second reagent vol. (µl)	0		
Sample Volume (µl)	3.0		
Lower normal value	@	Slope	1.00
Higher normal value	@	Intercept	0.00
Systematic dilution	1	Reaction delay	05:12
Dilution For Urine	1		
Low Reagent Blank	0.0000		
High Reagent Blank	0.1500		
Maximum Deviation	0.0100		

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Test	Ureia UV		
Type	Kin. 1	Primary Wavelength	340
Unit	mg/dL	High Wavelength	---
Number of decimals	0	Low Wavelength	---
Diluent	99 Diluent	Systematic serum blank	NO
		Depletion Limit	3.5000
Pre-washing	NO	T Tolerance Factor	1.0000
First reagent vol. (µl)	300	C Compensation Factor	1.0000
Second reagent vol. (µl)	0		
Sample Volume (µl)	3.0		
Lower normal value	15	Slope	1.00
Higher normal value	45	Intercept	0.00
Dilution factor	2		
Systematic dilution	1	Reaction delay	00:24
Dilution For Urine	40	Incubation	01:00
		Total	01:24
Low Reagent Blank	0.4000		
High Reagent Blank	3.5000		
Maximum Deviation	0.0100		

REVISÃO: 09/11