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Test name: AEO	Test: AEO			
Test Bar Code: @				
Test Type: End Point	Curve Type: Blanked Linear			
Units: UI/mL	No. of Decimal Places: 0			
Primary Wavelength: 540	Secondary Wavelength:			
Read Time/Interval: 20	Sample Blank: No			
Factor:				
Calibration Interval: 168 horas				
Normalization Interval: @				
No. of Calibrators: 2	No. of Replicates: 2			
Low Blank A Limit: -0.100	High Blank A Limit: 2.000			
Low A Limit: -0.100	High A Limit: 2.500			
Low Normal: 0	High Normal: 200			
Linearity Limit: 800	Curve S.D. Limit: 8.0			
Test Name: AEO	Test: AEO			
Test Bar Code: @				
Sample Volume: 3	Sample Diluent:			
Rerun Dilution Ratio: 1	Predilution Ratio: 1			
Reagent Diluent:				
	Reagent	Bar	Diluent	Lag
	Volume	Code	Volume	Time
Reagent 1	300	AE1A		120
Controls: @				

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Test name: ALT/GPT		Test: ALT	
Test Bar Code: @			
Test Type: Kinetic		Curve Type: Blanked Linear	
Units: U/L		No. of Decimal Places: 0	
Primary Wavelength: 340		Secondary Wavelength: 380	
Read Time/Interval: 60		Sample Blank: No	
Factor:			
Calibration Interval: @			
Normalization Interval: @			
No. of Calibrators: 2 #		No. of Replicates: 2	
Low Blank A Limit: 1.000		High Blank A Limit: 2.000	
Low A Limit: - 0.450		High A Limit: 2.000	
Low Normal: @		High Normal: @	
Linearity Limit: 400		Curve S.D. Limit: @	
Test Name: ALT/GPT		Test: ALT	
Test Bar Code: @			
Sample Volume: 25 µL		Sample Diluent:	
Rerun Dilution Ratio: 5		Predilution Ratio: 1	
Reagent Diluent:			
	Reagent	Bar	Diluent
	Volume	Code	Volume
			Lag
			Time
Reagent 1	250 µL	AL1A	60
Controls: @			

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Test name: AST/GOT	Test: AST		
Test Bar Code: @			
Test Type: Kinetic	Curve Type: Blanked Linear		
Units: U/L	No. of Decimal Places: 0		
Primary Wavelength: 340	Secondary Wavelength: 380		
Read Time/Interval: 60	Sample Blank: No		
Factor:			
Calibration Interval: @			
Normalization Interval: @			
No. of Calibrators: 3 #	No. of Replicates: 2		
Low Blank A Limit: 1.000	High Blank A Limit: 2.000		
Low A Limit: - 0.450	High A Limit: 2.000		
Low Normal: 0	High Normal: 42		
Linearity Limit: 400	Curve S.D. Limit: @		
Test Name: AST/GOT	Test: AST		
Test Bar Code: @			
Sample Volume: 25 µL	Sample Diluent:		
Rerun Dilution Ratio: 5	Predilution Ratio: 1		
Reagent Diluent:			
	Reagent	Bar	Diluent
	Volume	Code	Volume
			Lag
			Time
Reagent 1	250 µL	AS1A	60
Controls: @			

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Test name: Calcio	Test: Ca			
Test Bar Code: @				
Test Type: End Point	Curve Type: Blanked Linear			
Units: mg/dL	No. of Decimal Places: 1			
Primary Wavelength: 570	Secondary Wavelength: 600 nm			
Read Time/Interval: 20	Sample Blank: No			
Factor:				
Calibration Interval: @				
Normalization Interval: @				
No. of Calibrators: 2#	No. of Replicates: 2			
Low Blank A Limit: -0.100	High Blank A Limit: 0.750			
Low A Limit: 0.010	High A Limit: 1.800			
Low Normal: 8.8	High Normal: 11.0			
Linearity Limit: 16	Curve S.D. Limit: 0.500			
Test Name: Calcio	Test: Ca			
Test Bar Code: @				
Sample Volume: 6 µL	Sample Diluent:			
Rerun Dilution Ratio: 2	Predilution Ratio: 1			
Reagent Diluent:				
	Reagent	Bar	Diluent	Lag
	Volume	Code	Volume	Time
Reagent 1	225	CA1A		20 seg
Reagent 2	75	CA2A		60 seg
Controls: @				

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Test name: CHE	Test: CHE		
Test Bar Code: @			
Test Type: Kinetic	Curve Type: Enzyme Linear		
Units: U/L	No. of Decimal Places: 0		
Primary Wavelength: 405	Secondary Wavelength:		
Read Time/Interval: 180	Sample Blank: No		
Factor:			
Calibration Interval: @			
Normalization Interval: @			
No. of Calibrators: @	No. of Replicates: 2		
Low Blank A Limit: 0.100	High Blank A Limit: 2.000		
Low A Limit: - 0.200	High A Limit: 2.000		
Low Normal: 3930	High Normal: 11500		
Linearity Limit: 20.000	Curve S.D. Limit:		
Test Name: CHE	Test: CHE		
Test Bar Code: @			
Sample Volume: 4	Sample Diluent:		
Rerun Dilution Ratio: 5	Predilution Ratio: 1		
Reagent Diluent:			
	Reagent	Bar	Diluent
	Volume	Code	Volume
			Lag
			Time
Reagent 1	200	CH1A	180
Reagent 1	50	CH1A	120
Controls: @			

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Test name: CK	Test: CK		
Test Bar Code: @			
Test Type: Kinetic	Curve Type: Enzyme Linear		
Units: U/L	No. of Decimal Places: 0		
Primary Wavelength: 340	Secondary Wavelength: 700		
Read Time/Interval: 120	Sample Blank: No		
Factor:			
Calibration Interval: @			
Normalization Interval: @			
No. of Calibrators: 1	No. of Replicates: 2		
Low Blank A Limit: 0.000	High Blank A Limit: 0.500		
Low A Limit: 0.000	High A Limit: 1.800		
Low Normal: 26	High Normal: 189		
Linearity Limit: 2000	Curve S.D. Limit:		
Test Name: CK	Test: CK		
Test Bar Code: @			
Sample Volume: 5 µL	Sample Diluent:		
Rerun Dilution Ratio: 1#	Predilution Ratio: 1		
Reagent Diluent:			
	Reagent	Bar	Diluent
	Volume	Code	Volume
			Lag
			Time
Reagent 1	200	CK1A	10
Reagent 2	50	CK2A	120
Controls: @			

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Test name: CK-MB		Test: CK-MB		
Test Bar Code: @				
Test Type: Kinetic		Curve Type: Enzyme Linear		
Units: U/L		No. of Decimal Places: 0		
Primary Wavelength: 340		Secondary Wavelength: 380		
Read Time/Interval: 180		Sample Blank: No		
Factor:				
Calibration Interval: @				
Normalization Interval: @				
No. of Calibrators: 1#		No. of Replicates: 2		
Low Blank A Limit: 0.000		High Blank A Limit: 0.500		
Low A Limit: 0.000		High A Limit: 1.800		
Low Normal: 0		High Normal: 25		
Linearity Limit: 600		Curve S.D. Limit:		
Test Name: CK-MB		Test: CK-MB		
Test Bar Code: @				
Sample Volume: 12 µL		Sample Diluent:		
Rerun Dilution Ratio: 1		Predilution Ratio: 1		
Reagent Diluent:				
	Reagent	Bar	Diluent	Lag
	Volume	Code	Volume	Time
	Reagent 1	192	MB1A	120
	Reagent 2	48	MB2A	180
	Controls: @			

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Test name: Ferritina	Test: FERRI		
Test Bar Code: @			
Test Type: End Point	Curve Type: 2-Logit 5		
Units: µg/L	No. of Decimal Places: 0		
Primary Wavelength: 540	Secondary Wavelength: 0		
Read Time/Interval: 30	Sample Blank: No		
Factor:			
Calibration Interval: 168 horas			
Normalization Interval: @			
No. of Calibrators: 5	No. of Replicates: 2		
Low Blank A Limit: 0.000	High Blank A Limit: 2.000		
Low A Limit: -0.100	High A Limit: 2.500		
Low Normal: 20	High Normal: 250		
Linearity Limit: 500	Curve S.D. Limit: 8.0		
Test Name: Ferritina	Test: FERRI		
Test Bar Code: @			
Sample Volume: 8 µL	Sample Diluent:		
Rerun Dilution Ratio: 1	Predilution Ratio: 1		
Reagent Diluent:			
	Reagent	Bar	Diluent
	Volume	Code	Volume
			Lag
			Time
Reagent 1	250	FR1A	300
Controls: @			

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Test name: FERRO		Test: FERRO	
Test Bar Code: @			
Test Type: End Point		Curve Type: Blanked Linear	
Units: µg/dL		No. of Decimal Places: 0	
Primary Wavelength: 600		Secondary Wavelength:	
Read Time/Interval: 300		Sample Blank: No	
Factor:			
Calibration Interval: 168 horas			
Normalization Interval: @			
No. of Calibrators: 2		No. of Replicates: 2	
Low Blank A Limit: -0.100		High Blank A Limit: 2.000	
Low A Limit: -0.100		High A Limit: 2.000	
Low Normal: 55		High Normal: 155	
Linearity Limit: 500		Curve S.D. Limit: 10	
Test Name: FERRO		Test: FERRO	
Test Bar Code: @			
Sample Volume: 15		Sample Diluent:	
Rerun Dilution Ratio: 2		Predilution Ratio: 1	
Reagent Diluent:			
	Reagent	Bar	Diluent
	Volume	Code	Volume
			Lag
			Time
Reagent 1	300	FEC1	300
Controls: @			

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Express Plus: wash fluid: water

Test name: HDL DIRETO	Test: HDLD			
Test Bar Code: @				
Test Type: End Point	Curve Type: Blanked Linear			
Units: mg/dL	No. of Decimal Places: 0			
Primary Wavelength: 540	Secondary Wavelength: 600			
Read Time/Interval: 10	Sample Blank: Yes			
Factor:				
Calibration Interval: @				
Normalization Interval: @				
No. of Calibrators: 2#	No. of Replicates: 2			
Low Blank A Limit: - 0.010	High Blank A Limit: 0.500			
Low A Limit: 0.000	High A Limit: 2.000			
Low Normal: @	High Normal: @			
Linearity Limit: 200	Curve S.D. Limit: 5.0			
Test Name: HDL DIRETO	Test: HDLD			
Test Bar Code: @				
Sample Volume: 3 µL	Sample Diluent:			
Rerun Dilution Ratio: 1	Predilution Ratio: 1			
Reagent Diluent:				
	Reagent	Bar	Diluent	Lag
	Volume	Code	Volume	Time
Reagent 1	250	HD1A		300
Reagent 2	83	HD2A		290
Controls: @				

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Test name: LIP	Test: LIP			
Test Bar Code: @				
Test Type: Twopoint	Curve Type: Blanked Linear			
Units: U/L	No. of Decimal Places: 0			
Primary Wavelength: 570	Secondary Wavelength: No			
Read Time/Interval: 90	Sample Blank: No			
Factor:				
Calibration Interval: @				
Normalization Interval:				
No. of Calibrators: 2	No. of Replicates: 2			
Low Blank A Limit: 0.000	High Blank A Limit: 1.500			
Low A Limit: 0.000	High A Limit: 2.000			
Low Normal: @	High Normal: @			
Linearity Limit: 300	Curve S.D. Limit: 10.0			
Sample Volume: 4 µL	Sample Diluent:			
Rerun Dilution Ratio: 1	Predilution Ratio: 1			
Reagent Diluent :				
	Reagent	Bar	Diluent	Lag
	Volume	Code	Volume	Time
Reagent 1	225 µL			20 seg
Reagent 2	135 µL			90 seg
Controls: @				

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Test name: PCR	Test: PCR			
Test Bar Code: @				
Test Type: End Point	Curve Type: Blanked Linear			
Units: mg/L	No. of Decimal Places: 1			
Primary Wavelength: 540	Secondary Wavelength:			
Read Time/Interval: 20	Sample Blank: No			
Factor:				
Calibration Interval: 168 horas				
Normalization Interval: @				
No. of Calibrators: 2	No. of Replicates: 2			
Low Blank A Limit: -0.100	High Blank A Limit: 2.500			
Low A Limit: -0.100	High A Limit: 2.000			
Low Normal: 0	High Normal: 5.0			
Linearity Limit: 100	Curve S.D. Limit: 15.0			
Test Name: PCR	Test: PCR			
Test Bar Code: @				
Sample Volume: 3	Sample Diluent:			
Rerun Dilution Ratio: 1	Predilution Ratio: 1			
Reagent Diluent:				
	Reagent	Bar	Diluent	Lag
	Volume	Code	Volume	Time
Reagent 1	300	PC1A		120
Controls: @				

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Test name: Proteinuria	Test: PU			
Test Bar Code: @				
Test Type: End Point	Curve Type: Blanked Linear			
Units: mg/dL	No. of Decimal Places: 0			
Primary Wavelength: 600	Secondary Wavelength:			
Read Time/Interval: 20	Sample Blank: No			
Factor:				
Calibration Interval: @				
Normalization Interval: @				
No. of Calibrators: 2#	No. of Replicates: 2			
Low Blank A Limit: 0.170	High Blank A Limit: 0.290			
Low A Limit: 0.000	High A Limit: 1.500			
Low Normal: 0	High Normal: 15			
Linearity Limit: 100	Curve S.D. Limit: 2.5			
Sample Volume: 15 µL	Sample Diluent:			
Rerun Dilution Ratio: 5	Predilution Ratio: 1			
Reagent Diluent:				
	Reagent	Bar	Diluent	Lag
	Volume	Code	Volume	Time
Reagent 1	300 µL	PU1A		300
Controls: @				

REVISÃO: 03/11